

Following the discharge of your cat after I131 thyroid treatment, your cat will be mildly radioactive. Due to the isolation period, much of the radioactive iodine which was administered to your cat has been eliminated.

A certain amount of radioactivity will still be present in your cat's urine and stool, so it is wise to take some very basic and simple steps for added safety. Using Bind-It hand soap and spray is an easy way to ensure an added layer of safety is maintained.

The specialized ingredients in Bind-It products are designed to lock and "bind" to any radioactive iodine it comes in contact with. Once bound, the radioactive iodine suspends in solution, and is removed by rinsing or wiping away.

S4FE-D™

Feline Isolation Kit

For hand and isolation area
cleaning after I131 treatment



747 Herra St Elburn, IL 60119
(800)542-1123 (630)630-1000
sales@labtechinc.com
www.I131safety.com
www.labtechinc.com



Following the simple steps at the right will help eliminate contamination from your cat in your home.

Radioactive iodine is a safe and extremely effective procedure to cure your cat of hyperthyroidism. Following the steps outlined above is a simple way to ensure complete safety in your home.

If you have any questions about using S4FE-D™, feel free to call or email us at: 1-800-542-1123 or sales@labtechinc.com

We wish you and your cat many years of happiness together.

Hand washing -

Wash your hands with S4FE-D™ CBRN Decon Soap after touching your cat or handling your cat's litter box within the first 2 weeks after your it returns home.

S4FE-D™ CBRN Decon Soap is used just like ordinary liquid soap. Wet you hands with water. Pump one to two pumps of S4FE-D™ CBRN Decon Soap into the palm of your hand. Rub your hand together for 15-30 seconds creating a good lather. Rinse well.

S4FE-D™ CBRN Decon Soap is very mild, so it will not irritate your skin. It is safe for children to use as well. Children should be supervised and should avoid eye, nose or mouth contact while the soap is on their hands.

Litter box, etc. -

After emptying the litter box, spray the inside of the box with S4FE-D™ CBRN Decon spray. Wait for 30 seconds, then wipe dry with paper towels. Dispose of paper towels in a separate waste bag.

If you are using flushable kitty litter, the area inside the bowl of the toilet and rim should be sprayed with S4FE-D™ CBRN Decon spray. Wait for 30 seconds then flush and wipe the outside rim of the toilet with paper towels. Dispose of paper towels in a separate waste bag.

If your soiled litter gets outside the litter box or your cat urinated in any area outside the litter box, thoroughly spray that area with S4FE-D™ CBRN Decon spray. Wait for 30 seconds, then wipe dry with paper towels. Dispose of paper towels in a separate waste bag.

If the area is carpeted or is fabric (such as bed linens or furniture), soak well and blot thoroughly with paper towels to remove as much solution as possible. Dispose of paper towels in a separate waste bag.

Keep all used paper towels away from children and pets and dispose of promptly.